

Assistive Technology Training Online

University at Buffalo - The State University of New York
USDE# H324M980014
http://at-training.com



Using IntelliKeys USB with Switches

IntelliKeys ([IntelliTools](#)) can be used as a switch interface for up to 2 switches with software programs requiring a certain combination of key input. For 2 switches, the left side of the keyboard works as Switch #1 and the right, as Switch #2. This option provides a way for partial participation of a student in a group activity.

| | |
|-------|-----------------|
| CLICK | DOUBLE CLICK |
|-------|-----------------|

Refer to the Switch Settings Chart to select the level that contains the required keys.

Notice that IntelliKeys and IntelliKeys USB have different settings.

To setup IntelliKeys USB to be used with switches:

- Examine the software program to determine which keys the student will need.
- With IntelliKeys connected to the computer, slide in the **Setup Overlay**. Locate the **Trouble Shooting Section** and Press **Show Control Panel**



Shooting Section and Press **Show Control Panel**

- Click the **Advanced Tab** and select **As a switch with these settings**
 - From the list, select the setting that you want and click **Done**.
 - For example, if the software requires a mouse click and double-click, select Level 5 In IntelliKeys USB.
- Attach the switches to the switch ports on the left side of the keyboard and you are ready to go. Remember that the keyboard surface also acts as a two-switch interface.